DE/EN

**GEBRAUCHS
ANWEISUNG**
**INSTRUCTIONS
FOR USE**

RemoBOW

C.S.S.
MICRO SYSTEMS

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WEEE DE94635635

Table of Contents

1. Introduction	22
2. Intended use	22
3. Explanation of symbols	22
4. Safety instructions	23
5. Cleaning and disinfection	24
6. Scope of delivery	25
7. Device Buttons, Ports, Indicator	26
Color Table: Status LED	27
Color Table: LED Ring	27
8. Installation	28
Pairing RemoBOW ABL with bed	28
Pairing RemoBOW EW with bed	30
Connecting the AssistX Mobil call system	31
Pairing an Easywave adapter plug for a lamp	31
Mounting	32
9. Use of the Device	32
Switching on/off	32
Controlling the bed	32
Triggering an AssistX mobile call	33
Switching a lamp on and off	33
Charging the battery	34
10. Maintenance / Review	34
11. Support	35
12. Technical specifications	35
13. Conformity / Standards	36
14. Environmental protection & disposal	36

1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

2. Intended use

RemoBOW is used to control care beds with big buttons that can be easily operated even with motor or cognitive impairments. **RemoBOW** supports the direct control of AssistX Mobil call systems and an Easywave adapter for an external lamp. **RemoBOW** can be mounted directly on the care bed.

The "ABL" variant directly supports care beds from the manufacturers Burmeier and Stieglmeyer, which use a protected Bluetooth® control system. These manufacturers have expressly approved the combination of their corresponding beds with the **RemoBOW**.

Other nursing beds require a bed control interface with Easywave support (*RemoBIF* or *BIF8plus*)

3. Explanation of symbols

Safety instructions!

Important instructions!

4. Safety instructions

Handling

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification.

The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision.

Do not use the device in damp, wet, very cold or very hot environments (*see temperature information in technical data*).

Never replace the built-in batteries by yourself! The battery can be damaged, which can lead to overheating and can lead to injuries. The integrated rechargeable lithium polymer battery may only be replaced by authorized personal.

Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting. The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

Repair

Under no circumstances should you open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

Interaction with medical devices

The device and its components can emit electromagnetic radiation and have integrated magnets. The device must therefore not be placed on a person or brought closer than 15 cm to people using active medical devices use. The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers. If necessary, please contact the device manufacturer of the other active medical device.

5. Cleaning and disinfection

Before cleaning the device, disconnect all cables and switch off. The device should be cleaned if required with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.

Do not use sprays that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

6. Scope of delivery

- A) **RemoBOW**
- B) USB C power adapter
- C) Instructions for Use
- D) Magnet for handset (*variant ABL*)

A

B

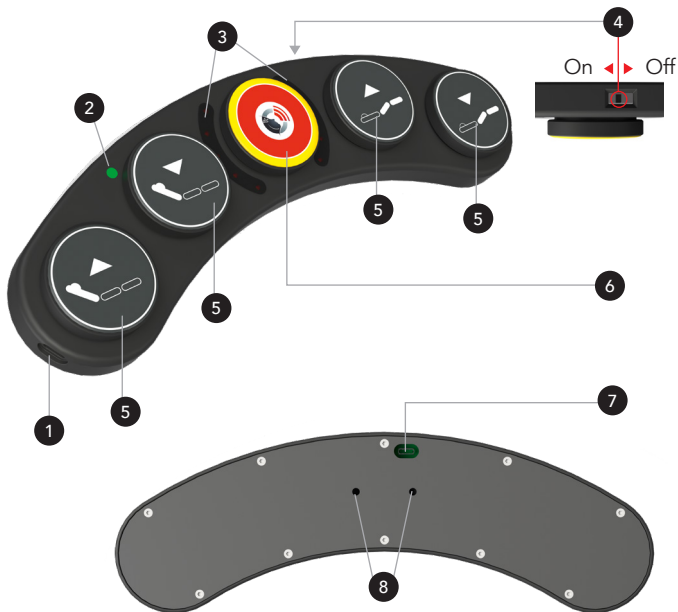
C

D

7. Device Buttons, Ports, Indicator

1 USB C charging socket 1

2 Status LED

3 LED Ring

4 On/Off switch

5 Buttons for bed control

6 Buttons for calling and lamp

7 USB C charging socket 2

8 Mounting threads M4
(distance 32 mm)

Color Table Status-LED 2

LED		Meaning
Green flashing		When button is pressed: battery fully charged
Red		Battery almost empty (< 20%)
Red flashing		When switching on: Error, e.g. ABL variant: not paired
Red flashing fast		When switching off for 15 seconds
Orange flashing		When button is pressed: battery 20-80% charged
Blue flashing		Pairing active (ABL variant)

EN

Color Table LED-Ring 3

LED		Meaning
Red		Reassurance light (with AssistX Mobil)
Green		Call confirmed (with AssistX Mobil)
Yellow		Button 6 pressed for lamp control
Blue		For ABL variant: When pressing bed control buttons: Bed not paired or not unlocked

8. Installation

Connect the USB-C power adapter to the charging socket **1** or **7** and charge the battery for at least 30 minutes.

Switching on: Switch the device on using the on/off slide switch **4** on the back (*slide to the right when viewed from above*). The LED **2** initially lights up green (*colors may vary depending on the status, see table at 7*).

Before use, the bed must be paired with **RemoBOW** once. With the **RemoBOW ABL** variant, the care bed (*Dali from Burmeier and other care beds from Stiegemeyer/Burmeier*) is paired directly, with the **RemoBOW EW** variant, the additionally required bed interface RemoBIF or BIF 8+ is connected via Easywave.

Pairing RemoBOW ABL with bed

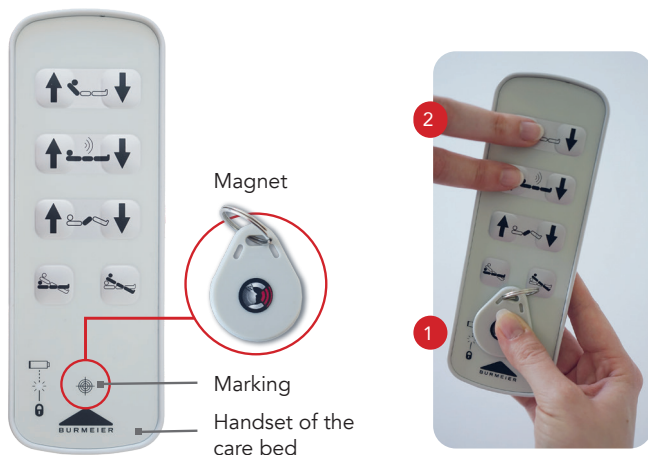
The **RemoBOW ABL** must be close to the bed control unit (*max. 1.5 m away*) during the entire pairing process.

- Switch off **RemoBOW**. To do this, press the on/off switch **4** at the back of the device (*slide to the left*). Wait approx. 15 seconds until the LED **2** (*flashing*) goes out.
 - Unplug the power supply of the care bed for at least 20 seconds (*until the green LED on the power supply no longer lights up*)
 - Plug in the bed's power supply and then switch on **RemoBOW** immediately afterwards (*within 5 seconds*) by pressing the on/off switch. After approx. 10 seconds, **RemoBOW** is paired with the bed.
-

- Unlock **RemoBOW** with handset:

Place the corresponding magnet (also included in the **RemoBOW** delivery scope) on the corresponding marking on the handset of the nursing bed **1** and press "Backrest up" and "Lying surface up" **2**.

EN

The bed can now be controlled directly with **RemoBOW**.

- Pairing the handset with the bed

If the handset of the care bed is not (*or no longer*) paired with the bed, unplug the bed's power supply for 10 seconds. Plug in the power supply and press "Bed up"  and "bed down"  simultaneously for at least 5 seconds within 20 seconds.

Note:

If **RemoBOW** is already paired with a bed and you want to pair with a new bed, make sure that the other bed is turned off (*power supply unplugged*). You can then pair the new bed as described above; the other bed will then no longer be connected. If the new pairing process is canceled or was not successful, the previous bed will remain paired.

Pairing RemoBOW EW with bed

In this variant, **RemoBOW** is connected to an additionally available bed control interface RemoBIF or BIF8+ via Easywave.

To do this, please follow the instructions for RemoBIF or BIF8+. To pair the respective function (*e.g. backrest up*) with the Easywave receiver in the bed control interface, set the Easywave receiver to the function in pairing mode. The appropriate Easywave signal is sent by **RemoBOW** when you press the corresponding button  (*backrest up/down, footrest up/down*). The function is now paired. This must be done for all desired functions.

The bed can now be controlled via **RemoBOW**.

Connecting the AssistX Mobil call system

You can pair **RemoBOW** directly with an AssistX Mobil receiver (*not included*, www.assistx.de). This means that a call can be triggered using the middle button **6**. To do this, press and hold the button **6**. On the AssistX mobile receiver, immediately press the „+“ button until the request to trigger the call appears. When the confirmation appears on the AssistX Mobil receiver, you can release the button **6**. **RemoBOW** is now paired with the AssistX Mobil receiver. A call can now be triggered by pressing the button **6** for a long time (*at least 3 seconds*). (Note: to confirm the call (LED ring **3** lights up green), press the button **6** for at least 3 seconds.)

Pairing an Easywave adapter plug for a lamp

To control a lamp, an Easywave adapter plug (RCP20) is required (*not included*). Plug the Easywave socket into the socket and go into pairing mode (*toggle*). To do this, please follow the instructions for the Easywave adapter plug.

The appropriate Easywave signal is sent by **RemoBOW** when you briefly press the button **6** (*max. 1 second*). The adapter plug is now paired with **RemoBOW**.

A lamp that is plugged into the adapter plug (*switch on the lamp using its own switch*) can now be switched on and off with the button **6** by briefly pressing it.

Mounting

RemoBOW can be mounted directly on the bed. There are mounting threads ⑧ (M4, distance 32 mm) on the back. It is recommended to position **RemoBOW** on the bed with suitable mount (*recommended by CSS*) so that the buttons are easily accessible.

If **RemoBOW** is mounted on a mount, it is recommended to use the USB-C charging socket 2 ⑦ and to lead the charger cable downwards along the mount. This avoids damaging the USB-C connector and the cable does not bother the user.

9. Use of the Device

Switching on/off:

Switching on: Switch the device on using the on/off slide switch ④ on the back (*slide to the right when viewed from above*). The LED ② initially lights up green (*colors may vary depending on the status, see table under 7*).

Switching off: To do this, use the on/off switch ④ on the back of the device (*slide to the left*). Wait approx. 15 seconds until the LED ② (*flashing*) has gone out.

After switching on, **RemoBOW** is ready for use after a short internal check when the LED ③ has gone out.

Controlling the bed

To control the backrest or footrest of the nursing bed, press the buttons ⑤ according to the symbol shown. The corresponding movement is made as long as the button is pressed, but for a maximum of 3 seconds (*safety function*). To continue the function, press the button again.

Note:

If the bed is not moving, please check whether the bed is correctly paired (see 8. *Installation*). With the ABL version, the LED ring ③ lights up blue in this case, meaning that the bed is either not paired or not unlocked.

Triggering an AssistX Mobil call

If **RemoBOW** is paired with an AssistX Mobil receiver, a call can be triggered using the button ⑥. To do this, press the button ⑥ for at least 3 seconds. The LED ring ③ lights up red as confirmation (*reassurance lamp*). If the call is confirmed by the caregiver/relative on the receiver, the LED ring ③ lights up green.

To finally confirm the call, the caregiver/relative must press the button ③ for at least 3 seconds.

Note: If the LED ring ③ does not light up after the call is triggered, the AssistX Mobil receiver is not switched on or is out of range. (See also the *AssistX Mobil Instructions for use*).

Switching a lamp on and off

If **RemoBOW** is paired with an Easywave adapter plug, a lamp connected to the adapter plug can be switched on and off by briefly pressing the button ⑥ (1 second). When the button ⑥ is briefly pressed, the LED ring ③ lights up yellow.

Charging the battery

RemoBOW has an internal battery with a usage-dependent life time of approx. 4 days. To charge, the supplied USB-C power supply must be connected to the USB-C charging socket ① or ⑦. If the battery is completely empty, the charging time is approx. 3 hours.

It is recommended to charge **RemoBOW** every 3 days.

The battery charge level is always briefly shown at the LED ② when a button is pressed (*green = 80-100% charged, orange = 20-80%, red less than 20%*). If the power supply is connected, the LED shows the status permanently.

Note: The power supply can also be permanently connected to **RemoBOW**. This will not damage the battery. Use original power supplies from CSS only!

Note: If an AssistX Mobil receiver is paired, **RemoBOW** automatically sends a corresponding call before the battery is completely empty.

10. Maintenance / Review

The device must be checked for safety and function at regular intervals of 3 months. The following checks must be carried out:

- function
 - plug connector for damage and contamination
 - damage to the casing and other components
-

11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to support@csslabs.de. Please be sure to include a description of the problem and the serial number.

12. Technical specifications

Dimensions: 300 mm x 65 (*wide of case, curved*) x 30 mm (*with buttons*)

Buttons: D 50 mm

Mounting: 2 threads M4 (*distance 32 mm/1¼"*)

Weight: 430 g / 450 g (*ABL*)

IP Rating: IP 54

RemoBOW ABL variant: for Burmeier "Dali" care bed or other Stieglmeyer/Burmeier care beds with corresponding equipment, Bluetooth® BLE interface (*only for the connection to the bed*).

RemoBOW EW variant: for controlling RemoBIF/BIF8+ bed interfaces via Easywave

Interfaces: Easywave, Bluetooth® BLE (*variant ABL*), AssistX radio

Power supply: 5V (*USB C*), typical 20 mA, max. 500mA

Battery life time: approx. 4 days (*dependent on usage*)

Temperature ranges

Use: 0 °C to 45 °C

Transportation and storage: -10 °C to 50 °C

13. Conformity / Standards

 This product is CE compliant.

The product variant "ABL" contains a LINAK ABL module.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG Inc. and any use of such marks and logos by LINAK® is under license.

14. Environmental protection & disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2012/19/EU and must therefore be properly disposed at the WEEE collection points.

WEEE DE94635635

ASSISTIVE TOOLS FOR A BETTER LIFE

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© 2024 CSS MicroSystems GmbH