DE/EN

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**INSTRUCTIONS  
FOR USE**

# AssistX ITX

**C.S.S.**  
MICRO SYSTEMS

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---

## Table of Contents

1. Introduction .....	20
2. Intended use .....	20
3. Explanation of symbols .....	20
4. Safety instructions .....	21
5. Cleaning and disinfection .....	22
6. Scope of delivery .....	23
7. Device Buttons, Ports, Indicator .....	24
Table Status LED power supply .....	25
Table Status LED at call/confirmation button .....	25
Table Status LEDs AssistX Ports .....	25
8. Installation .....	26
Pairing with AssistX Mobil receiver .....	26
Call triggering via infrared or Easywave .....	26
9. Use of the Device .....	27
Switching on/off .....	27
Triggering a call .....	27
Staff call .....	28
Priorities .....	28
Switch and sensor connection .....	30
Connection of ventilators and pulse oximeters via interface cable .....	30
10. Maintenance / Review .....	31
11. Support .....	32
12. Technical specifications .....	32
13. Conformity / Standards .....	33
14. Environmental protection & disposal .....	33

---

## 1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

## 2. Intended use

**AssistX ITX** is a multi-port transmitter for the AssistX Mobile call system, specially designed for use in long term intensive care. **AssistX ITX** has four patented AssistX Ports with automatic detection. Up to four control elements can be connected to it for call triggering or, via special interface cables, the device alarms of medical devices such as pulse oximeters and ventilators can be connected for call forwarding.

The connections are prioritized differently. Important call triggers, such as a ventilator, are prioritized on the receiver of the AssistX Mobile call system and displayed in the order of priority.

**AssistX ITX** has a special button for a staff call so that other nursing staff can be called in an emergency. This call has the highest priority.

## 3. Explanation of symbols

Safety instructions!

Important instructions!

---

## 4. Safety instructions

### Handling

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification.

The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision.

Do not use the device in damp, wet, very cold or very hot environments (see temperature information in technical data).

Never replace the built-in batteries by yourself! The battery can be damaged, which can lead to overheating and can lead to injuries. The integrated rechargeable lithium polymer battery may only be replaced by authorized personal.

### Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting. The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

## Repair

Under no circumstances you should open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

## Interaction with medical devices

The device and its components can emit electromagnetic radiation and have integrated magnets. The device must therefore not be placed on a person or brought closer than 15 cm to people using active medical devices use. The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers. If necessary, please contact the device manufacturer of the other active medical device.

## 5. Cleaning and disinfection

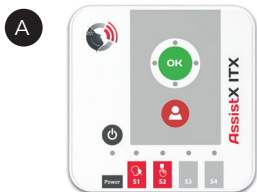
Before cleaning the device, disconnect all cables. The device should be cleaned if required with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.

**Do not use sprays** that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

---

## 6. Scope of delivery

- A) AssistX ITX
- B) USB-C power adapter
- C) Instructions for Use
- D) Wall mount (optionally)


### Optional accessories:

#20513 Wall mount

#20456 IR one infrared sensor

#20457 RX one Easywave radio receiver (not in all countries outside the EU)
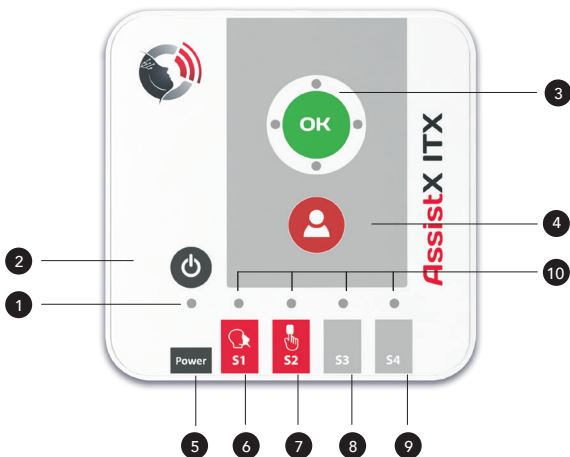
## 7. Device Buttons, Ports, Indicator

- 1 Status LED power supply
- 2 On/Off Button
- 3 Call / Confirmation button with status LEDs
- 4 Staff call button
- 5 Power supply 5V USB-C
- 6 AssistX Port\*, port for interfacecable ventilator
- 7 AssistX Port\*, port for interfacecable pulse oximeter
- 8 + 9 AssistX Ports, port for switches and sensors, IR/radio receivers
- 10 Status LEDs AssistX Ports

**1 Table Status LED power supply**

LED	Meaning
Green (dimly)	Normal operation, battery >80%
Green flashing	Battery operation, battery >80%
Yellow	Battery 20-80%, charging
Yellow flashing	Battery operation, battery 20-80%
Red flashing	Battery almost empty (connect power supply!)
flashing	Turn off (press and hold the On/Off button <b>2</b> for 5 sec)

**3 Table Status LED at call/confirmation button**

LED	Meaning
Red	Call activated and received (confirmation light)
Green	Call confirmed by care giver

**10 Table Status LEDs AssistX Ports**

LED	Meaning
Red flashing	2 sec while connecting and detecting device

## 8. Installation

Connect the USB-C power supply unit to the port ⑤ to supply the device with power and charge the internal battery. **AssistX ITX** is designed for operation with a power supply unit, the battery only serves as a backup (for at least 12 hours)

Switching on: Press the on/off button ② until the LED ① lights up dimly green.

**Note:** At least one AssistX Mobil receiver (not included) is required for operation. The AssistX Mobil receiver(s) must be equipped with at least firmware V2.00.

### Pairing with AssistX Mobil receiver

Press the “+” button on the AssistX Mobil receiver until the prompt to initiate a call appears. Now briefly press the button ③ on the **AssistX ITX**. The **AssistX ITX** is now paired with the AssistX Mobil receiver and ready for use.

**Test:** Press button ③, a call is now triggered on the receiver and the LEDs around the button ③ light up red. After confirmation on the receiver, the LEDs around ③ light up green. After pressing the button ③ again (confirmation), the LEDs around ③ go out.

**Note:** If the LEDs around ③ do not light up after a call is triggered, no AssistX Mobil receiver is paired, not switched on or out of range (see also the AssistX Mobile Instructions for Use).

### Call triggering via infrared or Easywave

If a call is to be triggered via an external device (e.g. communication device) using infrared or Easywave radio, connect the corresponding receiver (not supplied) IR one (infrared) or RX one (Easywave) to the connections ⑧ or ⑨ (not to ⑥ or ⑦).

---

## Mounting

It is recommended to mount **AssistX ITX** on the wall using. Other mounts, e.g. for standard rails, are available. Please ask your specialist reseller.

EN

## 9. Use of the Device

### Switching on/off:

**Switching on:** Press the On/Off button ② until the LED ① lights up dimly green. After switching on, **AssistX ITX** is ready for operation and the LED ① lights up dimly green.

**Switching off:** Press the On/Off button ② for at least 5 sec. until the LED ① is no longer lit. The device beeps twice as a confirmation.

### Triggering a call

A call is triggered by pressing a sensor or button connected to the **AssistX ITX** or by a connected medical device such as a ventilator or pulse oximeter. A call can also be triggered using button ③. A staff call can be triggered using button ④.

After triggering a call: The LEDs on button ③ light up red as confirmation (reassurance lamp). If the call is confirmed at the receiver by the carer/relative, you can recognize this by the fact that the LEDs at button ③ light up green.

To finally confirm the call, button ③ must be pressed by the carer/relative and the LEDs around ③ go out.

**Note:** If the LEDs around ③ do not light up after the call is triggered, no AssistX Mobil receiver is paired, it is not switched on or is out of range (see also the AssistX Mobil Instructions for use).

---

## Staff call

The button **4** can be used to trigger a staff call by care staff or relatives in order to be able to call other care staff in an emergency. This call has the highest priority. A blue flashing symbol with a yellow exclamation mark appears on all receivers, indicating the calling transmitter (in this case **AssistX ITX**). To avoid false alarms, the button (4) must be pressed for at least 2 seconds.

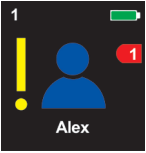
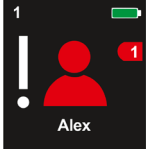
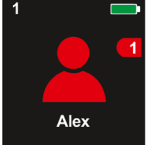
## Priorities

Calls that are triggered via the **AssistX ITX** have different priorities. This ensures that urgent calls to the AssistX Mobil are prioritized. If a normal call is active and a call with a higher priority is triggered, the prioritized call appears immediately on the receiver and the normal call is put on hold.

The connections **6**, **7**, **8** and **9** are identified by corresponding digits (S1, S2, S3, S4).

---

**Priorities:**

Priority 1:	Staff call via button <b>4</b>	
Priority 2:	Call via port <b>6</b> , usually a ventilator	
Priority 3:	Call via port <b>7</b> , usually a pulse oximeter	
Priority 4:	(normal call): Call via ports <b>8</b> or <b>9</b> or via button <b>3</b>	

## Switch and sensor connection

If a user can actively trigger a call, a corresponding button or sensor is simply connected to the connections 8 or 9. The AssistX Port\* automatically recognizes the call trigger, the corresponding LED 10 flashes red during the detection process (approx. 2 seconds). Do not press the button/sensor during this time!

If the button/sensor is unplugged during operation, the corresponding LED 10 flashes yellow and a corresponding call with the message "S" is shown on the Assist Mobil receiver as an alarm.

## Connection of ventilators and pulse oximeters via interface cable

Suitable interface cables are available for all common ventilators and pulse oximeters with call output for call systems. Please ask your specialist reseller or CSS MicroSystems for the correct cable.

Ventilators should be connected to port 6, pulse oximeters to port 7 (see priorities).

The AssistX Port\* automatically recognizes the interface cable, the corresponding LED 10 flashes red during the recognition process (approx. 2 seconds). Do not trigger an alarm during this time!

If the interface cable is disconnected during operation, the corresponding LED 10 flashes yellow) and a corresponding call with the message "S" is made on the Assist Mobil receiver as an alarm.

---

**Important!** Please note: connect the interface cable to the switched-on ventilator or pulse oximeter first, the device must be switched on in normal operation (no alarm), then connect to the AssistX ITX. The call function must then be tested by means of a forced alarm on the ventilator or pulse oximeter (e.g. disconnection).

## 10. Maintenance / Review

The device must be checked for safety and functionality at regular intervals of 3 months. The check must be done by trained specialist personnel only. The following should be checked:

- Battery (replace every 3 years at the latest)
- Range
- Connectors for damage and soiling
- Damage to the housing
- Update Firmware

**Please note:** If you update a device, all other components of the system (transmitters, receivers and repeaters) also need to be updated. Otherwise, safety-critical system failures can occur!

---

## 11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to [support@csslabs.de](mailto:support@csslabs.de). Please be sure to include a description of the problem and the serial number.

## 12. Technical specifications

**Dimensions:** 101 x 101 x 32 mm

**Weight:** 140 g

**IP Rating:** IP 54

**Interfaces:** 4x AssistX Port\* 3,5 mm jack sockets

**Power supply:** 5V (USB-C), max 500 mA

**Frequency:** 869,225 MHz

**Battery life time (backup):**  
more than 12 hours (dependent on usage)

**Temperature ranges:**  
Use: 0 °C to 45 °C  
Transportation and storage: -10 °C to 50 °C

---

\* Any sensors and switch can be connected via the 3.5 mm jack sockets using the patented **AssistX Port**. The control element is automatically detected and adapted accordingly. The power supply for active sensors and receivers (e.g. ProX proximity switch or AS one acoustic sensor) is also provided via this port, which you can recognize by the pink socket. The port is monitored and reacts to critical changes (e.g. cable break or permanent contact).

### 13. Conformity / Standards

 This product is CE compliant.

### 14. Environmental protection & disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points.

WEEE DE94635635

---

**ASSISTIVE TOOLS FOR A BETTER LIFE**

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© 2024 CSS MicroSystems GmbH