DE/EN

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**INSTRUCTIONS
FOR USE**

RemoBox

C·SS
MICRO SYSTEMS

DE Seite 3-26

EN Page 28-50

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WEEE DE94635635

Table of Contents

1. Introduction	29
2. Intended use	29
3. Explanation of symbols	30
4. Safety instructions	31
5. Cleaning and disinfection	34
6. Scope of delivery	35
7. Device Buttons, Ports, Display	36
8. Use of the device	38
9. Settings	39
Teaching in the radio code without a PC:	40
Teach-in radio code with PC:	42
10. Maintenance / Review	48
11. Support	48
12. Technical specifications	49
13. Conformity / Standards	49
14. Environmental protection & disposal	50

1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

2. Intended use

The **RemoBox** is an infrared radio converter. With infrared remote controls, e.g. environment control or communication devices with an integrated infrared transmitter, the complete range of ELDAT Easywave radio receivers can be used. The corresponding function of the **RemoBox** only needs to be taught into the Easywave radio receiver once.

3. Explanation of symbols

Safety instructions!

Important instructions!

4. Safety instructions

Handling

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification. The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision. Do not use the device in damp, wet, very cold or very hot environments (see temperature information in technical data).

Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting. The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

Repair

Under no circumstances you should open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

Interaction with medical devices

The device and its components can emit electromagnetic radiation and have integrated magnets. The device must therefore not be placed on a person or brought closer than 15 cm to people using other active medical devices. The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers. If necessary, please contact the device manufacturer of the other active medical device.

Attention! Magnets

The mounting plate contains a strong neodym magnet. Magnets can affect cardiac pacemakers and implanted defibrillators.

- The pacemaker could be switched into testmode and cause discomfort
- A defibrillator could stop working
- Keep enough distance to magnets if you use such devices
- Advice other users of such device against getting too close to magnets
- Keep magnets away from all devices and objects, which can be damaged by strong magnetic fields

Recommended safety distance is 6 inch (15 cm).

Magnets cause a far-reaching magnetic field. They could damage TV sets, laptops, PC harddrives, creditcards, ATM cards, data storage, mechanical watches, hearing aids and speakers.

Vehicle keys, USB sticks, cameras, cell phones and smart phones contain non-magnetic storage media. Therefore, static magnetic fields near those devices cannot delete data. There are countless covers with magnetic closures or magnetic holders on the market, which also speaks against the notion that magnets can damage these devices. However, it cannot be ruled out that very strong magnetic fields might magnetise and maybe damage mechanical parts or built-in speakers. When in doubt, keep these devices away from strong magnets.

5. Cleaning and disinfection

Before cleaning the device, switch it off and disconnect all cables. The device should be cleaned if required with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.


Do not use sprays that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

6. Scope of delivery

- **RemoBox**
- Plug-in power supply Mini USB
- USB cable Mini USB
- Instructions for Use

7. Device Buttons, Ports, Display

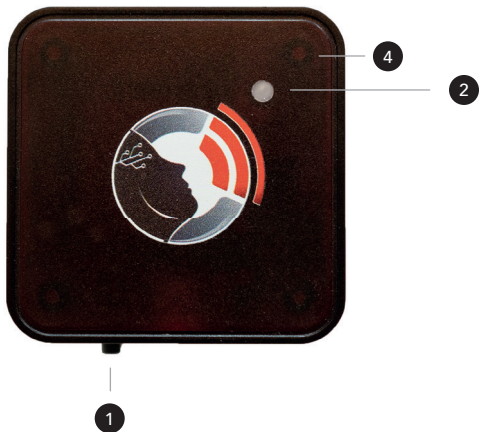-
- 1 Button
 - 2 LED
 - 3 Switch output
 - 4 Position of the infrared transmitter (on top of the housing)
 - 5 USB connector (power supply / configuration)
-

8. Use of the device

All ELDAT Easywave radio receivers can be controlled by the infrared radio converter **RemoBox** and, if required, this can be combined with Easywave handheld transmitters or with other **RemoBoxes**. In the case that only four radio receivers are required, the RemoBox can be switched to the IRS compatibility mode (see 9. Settings). In connection with a corresponding Easywave receiver, the **RemoBox** behaves like an older IRS infrared switch socket.

A contact socket with normally closed (NC) and normally open (NO) contacts located directly on the **RemoBox** can also be controlled directly via infrared, e.g. to activate a call system (see 9. Settings). The **RemoBox** can use simple PC software to generate all infrared codes for teaching into the corresponding environmental control device. So you don't need an additional remote control to teach in.

Functionality:

The **RemoBox** receives the infrared commands sent by a remote control and forwards them as radio codes to the Easywave receivers. The **RemoBox** can send 16 Easywave channels with up to 4 functions as radio codes to the receivers. Sockets only need 2 functions, so a

maximum of 32 sockets or 16 other actuators can be controlled. The radio receivers to be responded to must be taught in to the corresponding radio code of the **RemoBox**.

In addition to the radio transmitter, the **RemoBox** also has a potential-free switch contact in the form of a 3.5 mm stereo jack socket (attention: max. current load of 160 mA), which can be operated via an infrared remote control. All infrared codes for teaching into the corresponding environment control device can be generated using simple PC software. This software can also be used to send the required radio codes.

9. Settings

The settings can be made with a **RemoBox** remote control (optional, Art. #20139) or via the **RemoBox** Configurator software. You can also first teach-in the sockets (connect to Easywave receivers) and then teach-in the corresponding IR codes in the environmental control system.

Teaching in the radio code without a PC:

In order to be able to use the **RemoBox**, the Easywave receivers must be connected to it. Explanation here as an example with the Easywave RCP05 switch socket and its programming modes and operating modes: A total of three programming modes are available for the three operating modes of the socket-outlet receiver. The different modes are indicated by different flashing signals of the LED.

<i>Operating Mode</i>	<i>Function</i>	<i>LED Flashing</i>
1	ON/OFF (2 button operation)	
2	ON/OFF Toggle-Mode (1 button operation, press transmitter button > 1,6 sec)	
3	Timer Funktion 7 minutes, retrigger, (1 button operation)	

In the „Teach in transmitter“ mode, you transfer the codes of your transmitter buttons to the socket receiver:


**Note!**

When using 2 button operation, you only have to teach in one code. The receiver is linking the second code automatically.

1. Briefly press the programming button (< 1.6 sec). Programming mode 1 is activated, the LED flashes. If necessary, switch to another programming mode.
 2. Press the button on your transmitter that you use to control the device switch on and hold it down. If the Code has been saved, the ② LED lights up for approx. 4 sec and then flashes again. Release the transmitter button.
 3. Repeat the programming for your other transmitters. You can save max 32 transmitter codes in the device.
 4. Change by briefly pressing several times the ① Programming button in the operating mode.
-

Teach-in radio code with PC:

Here it is explained how the settings can be made with the RemoBox Configurator software (download free of charge at csslabs.de/download).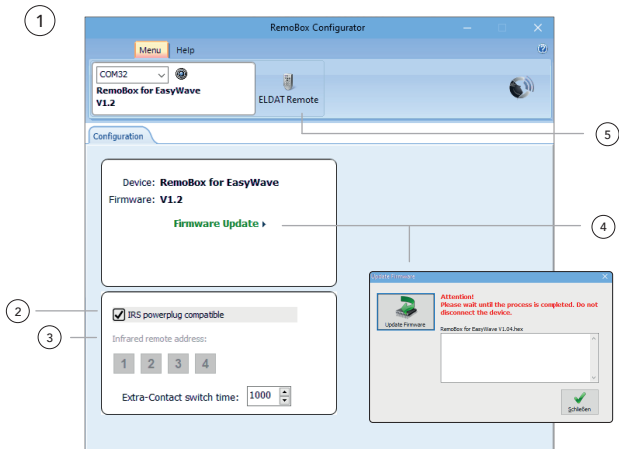

① **PC-Software**

The PC software is intended for Windows 8.1 or higher. During installation, the „RemoBox Configurator“ software and the required device drivers are installed.

After starting the software, you have to connect the RemoBox to a free USB port on your computer (USB cable is included). The RemoBox will then be recognized automatically. The PC software is almost self-explanatory.

② **IRS Infraredsocket compatible**

RemoBox may be used together with an ELDAT plug-in socket receiver like an IRS infrared socket system. In IRS mode, RemoBox works with four different addresses. So environmental controls that are programmed for the IRS system may keep on using their teached codes. For normal use do not activate this function!


③ **IR address (in Normal Mode)**

In normal mode, the address for infrared reception can be selected so that several independently addressable RemoBoxes can be used in the room. The switching time for the potential-free additional switching contact can also be set there. All settings made are immedia-

tely adopted by the RemoBox. **Do not** confuse the address of the infrared control with the Easywave channels!

④ **Update Firmware**

You will receive the latest firmware for the **RemoBox** with the latest version of the **RemoBox** Configurator. If there is a new firmware version available, you will see a message. To update you can click „Firmware Update“. In the window that opens, click on „Update Firmware“.
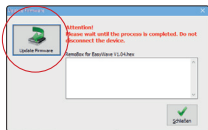


! Do not unplug the device during this process!
The device must be updated by the manufacturer.

Remote Control

A suitable remote control (Art #20139) is available separately for using the various codes. If you would like to use this, please read the relevant operating instructions for the remote control for further settings. If you do not want to use this, the screen remote control can be used, which is installed with the RemoBox Configurator software.

Screen Remote Control

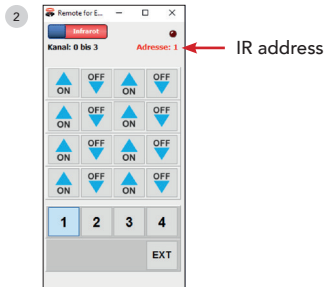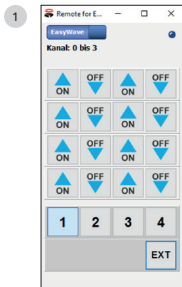- **1 Send radio commands**
(for teaching in Easywave receivers)

The “ELDAT remote control” button then takes you to the screen remote control. This window can be used to send the radio codes for set-up purposes at the push of a button. In the upper area, four functions (columns) can be used as function buttons for each channel (row). The channel range of the on-screen remote control is selected with the four lower range buttons. The selection is displayed accordingly at the top of the window.

Click the EXT- button to switch the RemoBox’s additional relay contact.

- **2 Send infrared codes**
(for teaching in environmental control devices)

The upper blue slider then takes you to the screen remote control for sending the corresponding infrared codes. This window can be used to reproduce the infrared codes for teaching in environmental control devices.

The buttons are structured according to the functions of the radio codes.

The built-in infrared transmitter is only intended for the purpose of teaching-in, so the teaching-in remote control should be held about 10 cm away from the RemoBox.

- ! It is important to know that the infrared signals send out from the top of the case. Turn accordingly if necessary and align in the direction of the learning device.
 - ! If a RemoBox is controlled in addition to the AssistX Mobil with IR from the communication device or environment control device, the first 4 Easywave functions conflict with IR channel 1 of the AssistX Mobil IR sensor. That is, e.g. the Easywave socket does not work on channel 1.
-

10. Maintenance / Review

The device usually does not require any maintenance. However, we recommend checking the following **every 3 months**:

- Functional test
- Connectors for damage and soiling
- Damage to the housing

11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to support@csslabs.de. Please be sure to include a description of the problem and the serial number of the device.

12. Technical specifications

Dimensions: 50 x 50 x 25,7 mm

Weight: 60 g

IP Rating: IP 54

Power supply: USB mini power supply

Connection: 3,5mm stereo jack socket, USB mini socket
switching current: max. 160 mA

Temperature ranges

Use: 0 °C to 45 °C

Transportation and storage: -10 °C to 50 °C

13. Conformity / Standards

 This product is CE compliant.

14. Environmental protection & disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points.

WEEE DE94635635

THE PASSIONATE ASSISTIVE TECHNOLOGY MANUFACTURER

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© 2021 CSS MicroSystems GmbH